



CORFLEX
.....

CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS



SHOULDER MISC

CRYOTHERM SHOULDER WRAP

- > Contours to shoulder, pockets hold gels packs to provide benefits of cold/hot therapy
- > Indicated for post-op use, treatment of arthritis or as part of a rehab program
- > Manufactured from soft breathable foam laminate with contact closure and pockets to hold reusable gel packs



PRODUCT NUMBER

0 GELS	2 GELS	4 GELS
72-7030	72-7032	72-7034

SIZING GUIDE

UNIV

Fits up to 48" chest circumference

Additional gels sold separately:
72-7001 - Single 5 1/2" x 11" Gel
72-7000 - Case/10 5 1/4" x 11" Gel

1. Remove two gel packs from freezer/microwave.
2. Observe warnings and instructions for use of gel pack.
3. Place one gel pack into each pocket and close with Velcro closure.
4. Locate "neck" area of wrap and place on the affected shoulder of patient.
5. Place short strap under upper arm and attach to wrap in the front and back.
6. Locate longer strap and attach to front of wrap near neck area so strap comes across chest under the arm of the unaffected shoulder, across back and attached to the back of wrap.



669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.