



CORFLEX®

Designed for life.

Universal Plantar Fasciitis Support

APPLICATION INSTRUCTIONS

Fitting and application:

Note: The brace comes packaged to be worn on the right ankle. See step #5 to prepare brace for use on left ankle.

- 1.** Detach 1½" elastic tension strap from brace body. Center the contoured notch of the wrap over the Achilles tendon at the lower portion of the leg.
- 2.** Wrap long end of brace body across front of ankle. Grasp short end of brace body with two hook closures and press to fasten snugly on long end.
- 3.** Wrap the elastic tension strap under the back edge of the arch and pull up and slightly back with both ends, press hook onto body wrap.
- 4.** Strap should apply pressure at back of arch just forward of heel.
- 5.** To prepare brace for left ankle detach 1½" elastic tension strap. Open two hook closures from body wrap. Fully remove both sides of each hook tip, rotate 180° and replace in same location.



Care instructions:

The Universal Plantar Fasciitis Support may be hand washed using warm water and detergent for delicate fabrics. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer.

Caution:

This product has been designed and tested based on single patient usage. Since it is a single use device it should never be used by multiple patients. If additional pain or symptoms occur while using this device, seek medical attention.

Indications:

Mild to moderate plantar fasciitis, heel pain and inflammation.

Contraindications:

Not for use on patients with circulatory or sensation issues. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use and consult your medical professional.

Warranty:

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to mis-use or application the warranty will be voided at the discretion of Corflex.

To Reorder:

Product Number

	Sm/Med	Lg/X-Lg
Each	61-1000	61-1001
10/pkg	61-1002	61-1003

Sizing Guide

Men	Up to 8	8.5+
Women	Up to 8.5	9+

Measurement denotes shoe size.



RELATED CORFLEX PRODUCTS

Ultra Stretch Plantar Fasciitis Splint

- Bi-lateral straps provide a controlled amount of dorsiflexion; removable wedge for added toe extension
- Indicated for treatment of plantar fasciitis, Achilles tendonitis, plantar flexion contractures, heel spurs and other lower leg over-use injuries
- Polyethylene shell with foam liner and padded straps with quick-release buckles; available with optional textured vinyl tread



	S	M	L	XL
	37-4191	37-4192	37-4193	37-4194
w/Tread	37-4291	37-4292	37-4293	37-4294

CORFLEX, INC.
669 INDUSTRIAL PARK DRIVE
MANCHESTER, NH 03109

DESIGNED FOR LIFE.
WWW.CORFLEX.COM
TOLL FREE: 800 426 7353