



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



CERVICAL SUPPORTS

PHILADELPHIA CERVICAL COLLAR W/TRACHEA

- > Features two-piece design anatomically contoured to provide superior stability and comfort with large trachea opening for easy access
- > Indicated for immobilization for both extrication and rehabilitation, post-operative discectomy and spinal fusion
- > Constructed of durable plastic laminated to soft hypoallergenic foam with contact closure



PRODUCT NUMBER

	PEDI	S	M	L
2 ¼"	19-4100	19-4121	19-4122	19-4123
3 ¼"		19-4131	19-4132	19-4133
4 ¼"		19-4141	19-4142	19-4143
5 ¼"		19-4151	19-4152	19-4153

SIZING GUIDE

PEDI	S	M	L
8"-11"	10"-13"	13"-16"	16"-19"

Measurement taken from circumference of the neck at "Adams apple"
Height measurement taken from posterior portion of mandible to the clavicle directly below



APPLICATION INSTRUCTIONS

Proper application of the Collar is as important as proper sizing for immobilization and patient comfort. The two-piece design consists of a FRONT piece and BACK piece which are packaged as a set.

Always maintain the patient's head in neutral alignment. These directions are guidelines only; follow a qualified healthcare provider's instructions.

1. After selecting the properly sized Collar, apply the BACK piece of the Collar to the back of the patient's neck. Center the Collar. The back arrow should point upward.
2. Apply the FRONT piece of the Collar with the chin secured in the recess. Center the Collar and ensure neutral alignment. The FRONT piece OVERLAPS the BACK piece to ensure effective immobilization and comfort. The front arrow should point upward.
3. With Velcro hook and loop fasteners, tighten the Collar with a bilateral adjustment. This will secure the patient's cervical region in neutral alignment.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.