



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**CERVICAL SUPPORTS**

**AMBU PERFIT COLLAR**

- > Features one-piece flat design with flip out chin piece and trachea opening for easy access
- > Indicated for immobilization for both extrication and rehabilitation
- > Constructed of durable plastic laminated to foam with contact closure



**PRODUCT NUMBER**

INFANT	PEDI	NECKLESS	SHORT	REGULAR	TALL
19-9001	19-9002	19-9003	19-9004	19-9005	19-9006

ACE ADJ	MINI ADJ
19-9007	19-9008

**SIZING GUIDE**

INFANT	PEDI	NECKLESS	SHORT	REGULAR	TALL
Purple	Aqua	Red	Yellow	Blue	Pink

ACE ADJ	MINI ADJ
Univ Adult	Univ Pediatric

**APPLICATION INSTRUCTIONS**

Proper application of any extrication collar requires two individuals. The first rescuer should maintain stabilization and neutral alignment of the head and neck by grasping the head with a spread finger technique.

1. The second rescuer should slide the front of the collar along the patient's chest and position the chin piece. The collar body should rest on the top of the patient's shoulder and against the sternum without gaps. The patient's chin must rest securely in the chin piece in neutral alignment position.
2. While holding the front of the collar in place, wrap the back of the collar around the back of the patient's head and neck and secure to the front of the collar with the Velcro tab.)
3. Adjust the Velcro tab as necessary by holding the front cutout of the collar and tightening the Velcro tab.
4. Adjustment of the size of the collar may be done on the patient as per your local protocol.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:

<b>TELEPHONE</b>	<b>FAX</b>	<b>E-MAIL</b>	<b>MAIL</b>
1-800-426-7353, Ext1	1-603-623-4111	Service@corflex.com	CORFLEX, Inc. Attn: Customer Service 669 East Industrial Park Dr Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.