



**CORFLEX.**  
.....

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## ARM SLINGS

### MESH ARM SLING

- > Allows for slide buckle or contact closure
- > Indicated for injuries to the arm, shoulder, wrist or hand and cast support to maintain arm in neutral position
- > Manufactured from seamless breathable mesh material



#### PRODUCT NUMBER

XS	S	M	L	XL
21-1730	21-1731	21-1732	21-1733	21-1734

#### SIZING GUIDE

	XS	S	M	L	XL
Width/Height	12" x 5"	15" x 6"	17" x 7"	19" x 8"	21" x 9"

Measurements are sling pouch width x height

### APPLICATION INSTRUCTIONS

1. Open sling and place arm inside, making sure elbow is in sling as far back as possible.
2. Thread strap through ring/buckle.
3. Secure hook and loop so elbow is positioned at 90-degree angle.
4. If applicable, position elbow (shoulder) strap and pull behind your back.
5. Thread strap through buckle and pull tight to prevent arm from moving away from body.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.