



**CORFLEX.**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## ARM SLINGS

### CRADLE ARM SLING

- > Open-end design allows arm to easily slide into sling
- > Indicated for injuries to the arm, shoulder, wrist or hand and cast support to maintain arm in neutral position
- > Manufactured from seamless durable material



### PRODUCT NUMBER

S	M	L
21-1976	21-1977	21-1978

### SIZING GUIDE

	S	M	L
Width/Height	11" x 9"	12" x 10"	13" x 11"

Measurements are sling pouch width x height

### APPLICATION INSTRUCTIONS

1. Place arm in sling so that it is centered between elbow and fingers.
2. Thread strap through d-ring.
3. Attach hook and loop closure to keep elbow at 90 degrees.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



#### TELEPHONE

1-800-426-7353, Ext1



#### FAX

1-603-623-4111



#### E-MAIL

Service@corflex.com



#### MAIL

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.