



**CORFLEX.**  
.....

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## ARM SLINGS

### COLLAR & CUFF

- > Features tubular neck pad and flannel lined cuff for optimal patient comfort
- > Indicated for injuries to the arm, shoulder, wrist or hand
- > Constructed of 1" durable strap with contact closure wrist cuff



### PRODUCT NUMBER

**UNIV**  
21-2052

### SIZING GUIDE

Fits up to a 6"-10" wrist circumference

### APPLICATION INSTRUCTIONS

1. Grasp collar strap and place around neck, making sure to center tubular pad on back of neck.
2. Slide collar strap thru buckle and secure.
3. Place cuff portion around wrist of affected arm and secure.
4. Adjust collar strap to keep elbow at 90 degrees.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**  
1-800-426-7353, Ext1



**FAX**  
1-603-623-4111



**E-MAIL**  
Service@corflex.com



**MAIL**  
CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.