



**CORFLEX.**  
.....

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## ARM SLINGS

### STRAP ARM SLING

- > Features tubular neck pad for added comfort and slide buckle for maximum adjustability
- > Indicated for injuries to the arm, shoulder, wrist or hand
- > Manufactured from 1½" heavy duty web strap material



### PRODUCT NUMBER

S	M	L	XL
21-5051	21-5052	21-5053	21-5054

### SIZING GUIDE

S	M	L	XL
30"-42"	40"-52"	50"-62"	60"-72"

Measurements are minimum / maximum adjustable strap length

### APPLICATION INSTRUCTIONS

1. Grasp strap at mid-point and place strap around neck, making sure tubular foam pad is center in back of neck.
2. Pass arm thru straps, making sure that one strap supports the wrist and the other supports arm at the mid-point.
3. Adjust buckle closures to keep elbow at 90 degrees.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.