



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



SHOULDER IMMOBILIZERS

ECONOMIZER SHOULDER IMMOBILIZER

- > Features 1" shoulder and waist straps with slide buckle
- > Indicated for immobilization following rotator cuff repairs, S/P glenohumeral acromioplasty surgery, or other shoulder related injuries
- > Manufactured from seamless lightweight muslin material



PRODUCT NUMBER

XS	S	M	L	XL
23-1740	23-1741	23-1742	23-1743	23-1744

SIZING GUIDE

	XS	S	M	L	XL
Width/Height	12" x 5"	15" x 6"	17" x 7"	19" x 8"	21" x 9"

Measurements are sling pouch width x height



APPLICATION INSTRUCTIONS

1. Open sling and place arm inside, making sure elbow is in sling as far back as possible.
2. Thread strap through ring/buckle.
3. Secure hook and loop so elbow is positioned at 90-degree angle.
4. If applicable, position elbow (shoulder) strap and pull behind your back.
5. Thread strap through buckle and pull tight to prevent arm from moving away from body.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.