



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



SHOULDER IMMOBILIZERS

UNIVERSAL SHOULDER IMMOBILIZER

- > Features 8" wide chest band with anti-migration strap and 2½" wide moveable humeral and wrist cuffs for ease of application; universal design allows for inventory reduction
- > Indicated for immobilization following rotator cuff repairs, S/P glenohumeral acromioplasty surgery, or other shoulder related injuries
- > Constructed of foam laminate to pressure sensitive fabric



PRODUCT NUMBER

UNIV
23-1750

SIZING GUIDE

UNIV
Fits 24" - 48" chest circumference



APPLICATION INSTRUCTIONS

1. Open up immobilizer, remove and set aside long shoulder strap, humeral cuff and wrist/forearm cuff.
2. Grasp widest portion of immobilizer, wrap around waist and secure hook and loop closure.
3. Secure humeral cuff (longer cuff) onto body of immobilizer just above elbow, place upper arm in cuff and secure.
4. Secure wrist/forearm cuff (smaller cuff) onto body of immobilizer in front of body, place forearm in cuff and secure.
5. Attach buckle-end of long shoulder strap to front portion of immobilizer, bring over unaffected shoulder, secure to back of immobilizer and adjust strap to desired tension level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.