



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## SHOULDER IMMOBILIZERS

### Shoulder Abduction Pillow w/Sling

Features easy to apply design with V-lock strapping system to prevent rolling and shifting

Indicated for treatment after Bankart repairs, rotator cuff repairs, shoulder arthroscopy, separations and dislocations

Constructed of firm density foam covered in tricot material with tricot sling and exercise ball to stimulate circulation



#### PRODUCT NUMBER

	S	M	L	XL
Standard	23-1911	23-1912	23-1913	23-1914
Firm Fit	23-1931	23-1932	23-1933	23-1934
Neutral	23-1921	23-1922	23-1923	23-1924

#### SIZING GUIDE

	S	M	L	XL
Hip	24"-30"	30"-36"	36"-42"	42"-48"
Forearm	9"-12"	12"-15"	15"-18"	18"-20"

Measurement taken from hip circumference and length of forearm from wrist to elbow

NOTE: Sling may be purchased separately

### APPLICATION INSTRUCTIONS

1. Separate sling from pillow and remove 1 1/2" x 16" secure strap from sling.
2. Detach shoulder strap and open front panel.
3. Place thumb through loop inside sling for additional comfort.
4. Pass shoulder strap diagonally up and over the unaffected shoulder and fasten through both buckles so the end of the strap is at mid chest. The shoulder strap length can also be adjusted at the back of the sling.
5. Place pillow at waistline with large end facing forward. Change angle of pillow by bending malleable stay if desired. Attach plastic buckle and adjust length of the strap at the back end of pillow. Attach sling to the pillow, lining up hook and loop contact closure for a secure fit.

Note: The Standard and Firm Fit pillows are designed to keep the arm in approximately 15 degree of abduction. The position of the pillow at the waist should be positioned for desired internal or external rotation by sliding the pillow forward or back along the waistline.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.