



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



SHOULDER IMMOBILIZERS

UNIVERSAL COMFORT SHOULDER IMMOBILIZER

- > Features 6" wide tricot chest band and padded humerus and forearm cuff for optimal patient comfort; universal design allows for inventory reduction
- > Indicated for immobilization following rotator cuff repairs, S/P glenohumeral acromioplasty surgery, or other shoulder related injuries
- > Manufactured from tricot material with polyester kodel pile liner on cuffs with contact closure



PRODUCT NUMBER

UNIV
23-4100

SIZING GUIDE

UNIV
Fits 22" - 45" chest circumference

APPLICATION INSTRUCTIONS

1. Open up immobilizer, release all hook and loop closures, remove and set aside humeral cuff and long shoulder strap.
2. Grasp widest portion of immobilizer, wrap around waist and secure hook and loop closure, making sure padded wrist/forearm cuff is in front of body.
3. Secure humeral cuff onto body of immobilizer just above elbow, place upper arm in cuff and secure.
4. Open padded wrist/forearm cuff, place forearm in cuff and secure.
5. Attach long shoulder strap to front portion of immobilizer, bring over unaffected shoulder and secure to back of immobilizer.

NOTE: any excess material may be trimmed if desired.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.