



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



SHOULDER IMMOBILIZERS

UNIVERSAL SLING & SWATHE

- > Features cradle-style sling with trimmable swathe for optimal fit; universal design allows for inventory reduction
- > Indicated for immobilization following rotator cuff repairs, S/P glenohumeral acromioplasty surgery, or other shoulder related injuries
- > Constructed of foam laminate to pressure sensitive fabric to optimize patient fit



PRODUCT NUMBER

UNIV
23-4204

SIZING GUIDE

UNIV
24"-48"

Measurement taken from chest circumference

APPLICATION INSTRUCTIONS

1. Separate sling from swathe and put swathe to one side.
2. Grasp sling and place arm in sling so that it is centered between elbow and fingers (Note: metal slide buckle and sew-in label should be facing outward).
3. Release hook closure on straps and place straps over each shoulder.
4. Grasp right strap and bring across center of back and under left side of sling. Secure hook to sling.
5. Grasp left strap and bring across center of back and under right side of sling. Secure hook to sling.
6. Adjust metal sling buckles on sling to keep elbow at 90 degrees.
7. Grasp swathe, attach swathe to hook on front of sling, wrap around body over affected arm and secure.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.