



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



SHOULDER IMMOBILIZERS

ULTRA SLING & SWATHE

- > Features high profile sling designed to encompass the elbow and swathe with contact closure
- > Indicated for immobilization following rotator cuff repairs, S/P glenohumeral acromioplasty surgery, or other shoulder related injuries
- > Manufactured from seamless, lightweight, durable material



PRODUCT NUMBER

S	M	L	XL
23-4401	23-4402	23-4403	23-4404

SIZING GUIDE

	S	M	L	XL
Sling Pouch	9"-10"	13"-10"	16"-11"	19"-12"
Chest Cir	24"-36"	30"-42"	36"-48"	42"-54"

APPLICATION INSTRUCTIONS

1. Place affected arm inside sling, making sure elbow is in sling as far back as possible.
2. Thread strap through buckle.
3. Adjust strap so elbow is at a 90-degree angle.
4. Attach swathe to hook on front of sling and wrap around body over affected arm and secure.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.