



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



SHOULDER IMMOBILIZERS

ELASTIC SHOULDER IMMOBILIZER

- > Features 4" wide elastic chest band and humerus and forearm cuff with contact closure
- > Indicated for immobilization following rotator cuff repairs, S/P glenohumeral acromioplasty surgery, or other shoulder related injuries
- > Constructed of premium quality elastic with vinyl cuffs



PRODUCT NUMBER

XS	S	M	L	XL	2XL
23-5240	23-5241	23-5242	23-5243	23-5244	23-5245

SIZING GUIDE

XS	S	M	L	XL	2XL
18"-24"	24"-30"	30"-36"	36"-42"	42"-48"	48"-54"

Measurements taken from chest circumference

APPLICATION INSTRUCTIONS

1. Attach 4" elastic strap so that humeral cuff aligns with affected arm.
2. Attach humeral cuff to affected arm just above elbow.
3. Attach other cuff snugly in the wrist area.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.