



CORFLEX.
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



SHOULDER MISC

ULTRA CLAVICLE STRAP

- > Features heavy-duty felt center point pad with hook and loop closure for optimal comfort
- > Indicated for clavicle fractures, Grade II/III acromioclavicular separations, sternoclavicular dislocations
- > Constructed of breathable, open cell stockinette covered foam with contact closure



PRODUCT NUMBER

2XS	XS	S	M	L	XL
25-1729	25-1730	25-1731	25-1732	25-1733	25-1734

2XL

25-1735

SIZING GUIDE

2XS	XS	S	M	L	XL
15"-21"	17"-24"	25"-31"	27"-33"	29"-35"	31"-37"

2XL

37"-42"

Measurement taken from center back, over shoulder under arm and back to center point of back

APPLICATION INSTRUCTIONS

Note: Application of the clavicle strap requires two people.

1. Place felt/foam pad in center of back.
2. Extend straps-one over left shoulder and one over right shoulder.
3. Grasp straps, slide through buckles and pull gently.
4. Secure hook to loop to secure to desired tension.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.