



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



BINDERS / BELTS

ELASTIC RIB BELT

- > Anatomically designed (male or female) to provide optimal comfort and support
- > Indicated for fractured or bruised ribs, intercostal muscle strains and costovertebral sprains
- > Manufactured from premium quality elastic with contact closure



PRODUCT NUMBER

	S	M	L	XL	2XL
Male	31-1901	31-1902	31-1903	31-1904	31-1905
Female	31-1911	31-1912	31-1913	31-1914	31-1915

SIZING GUIDE

	S	M	L	XL	2XL
	24"-30"	30"-36"	36"-42"	42"-48"	48"-54"

Measurement taken from chest circumference

APPLICATION INSTRUCTIONS

1. Open rib belt.
2. Grasp elastic and foam sides and position over rib area.
3. Gently stretch rib belt and secure hook portion along any portion of the foam liner.
4. If applicable, for elastic material, secure hook to loop portion for closure.

Note: Rib belt should not be worn for extended periods of time.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.