



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**BINDERS / BELTS**

**UNIVERSAL ABDOMINAL BINDER**

- > Features solid panel elastic for uniform support of entire abdominal region; universal design allows for inventory reduction
- > Indicated for abdominal muscle strain or post-op application
- > Constructed of plush elastic with soft foam laminate and contact closure



**PRODUCT NUMBER**

WIDTH	UNIV
9"	32-1950
12"	32-1951

**SIZING GUIDE**

UNIV
24"-48"

Measurement taken from chest circumference

**APPLICATION INSTRUCTIONS**

1. Open abdominal binder.
2. Grasp elastic and foam sides and position over abdominal area.
3. Gently stretch abdominal binder and attach hook portion along any portion of the foam liner.
4. If applicable, for elastic material, attach hook to loop portion for closure.

Note: Abdominal binder should not be worn for extended periods of time.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.