



CORFLEX
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



BINDERS / BELTS

PANEL ELASTIC ABDOMINAL BINDER

- > Features panel elastic construction with faggoted stitching between panels to prevent rolling; available in 9" or 12" width
- > Indicated for abdominal muscle strain or post-op application
- > Manufactured from premium panel elastic with contact closure



PRODUCT NUMBER

WIDTH	S	M	L	XL	2XL	3XL
9"	32-1961	32-1962	32-1963	32-1964	32-1965	32-1966
12"	32-1971	32-1972	32-1973	32-1974	32-1975	32-1976

WIDTH	4XL
9"	32-1967
12"	32-1977

SIZING GUIDE

S	M	L	XL	2XL	3XL
24"-30"	30"-36"	36"-42"	42"-48"	48"-54"	54"-60"

4XL
60"-66"

Measurement taken from chest circumference

APPLICATION INSTRUCTIONS

1. Open abdominal binder.
2. Grasp elastic and foam sides and position over abdominal area.
3. Gently stretch abdominal binder and attach hook portion along any portion of the foam liner.
4. If applicable, for elastic material, attach hook to loop portion for closure.

Note: Abdominal binder should not be worn for extended periods of time.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.