



**CORFLEX.**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**HIP MISC**

**TROCHANTER BELT**

- > Designed to provide support and stability over the sacral region
- > Indicated for instability of the sacroiliac joints and the lumbosacral joint
- > Constructed of soft pile laminated to flannel lining with elastic side pulls and contact closure



**PRODUCT NUMBER**

**UNIV**  
35-1419

**SIZING GUIDE**

**UNIV**  
Fits up to 48" hip circumference

**APPLICATION INSTRUCTIONS**

1. Open belt and lay flat.
2. Grasp ends of belt and position over center of hips.
3. Pull closed and secure with hook and loop closure.
4. Grasp both ends of elastic double pull, pull firmly and attach to the side for desired tension and support.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.