



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



BACK SUPPORTS

THORACOLUMBAR SUPPORT

- > Features convenient wrap-around design with suspenders for ease of application and double-pull tension straps for added compression, posterior pocket accepts thermoplastic insert (sold separately)
- > Indicated for pain in the thoraco lumbar region and compression fractures
- > Manufactured from premium quality elastic with adjustable suspenders and contact closure



Insert option:
37-6032 TLSO Thermoplastic Insert

PRODUCT NUMBER

XS	S	M	L	XL
35-1700	35-1701	35-1702	35-1703	35-1704

2XL

35-1705

SIZING GUIDE

XS	S	M	L	XL
18"-24"	24"-30"	30"-36"	36"-42"	42"-48"

2XL

48"-52"

3XL

52"-56"

4XL

56"-60"

Measurement taken from hip circumference

APPLICATION INSTRUCTIONS

1. Release suspender straps, open back support and lay flat.

Note: if applicable, heat mold thermoplastic insert separately to patient and then place molded insert into back pocket.

2. Grasp sides of support and center on small of back.
3. Pull left side of support firmly across abdominal region, repeat for right side and secure loop portion onto hook portion.
4. Grasp both ends of elastic double-pull, pull left side across and attach onto loop, then pull right side across and attach to loop.
5. Bring suspender straps over each shoulder and under arm.
6. Slide straps through plastic buckles in back and secure.
7. Adjust straps to desired tension level, making sure not to over tighten.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.