



CORFLEX
.....


**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



BACK SUPPORTS

V-LOCK BACK SUPPORT

- > Features double elastic v-shaped pull for easy application with contoured front for optimal patient comfort; also includes plastic and optional aluminum stays for flexible or firm support
- > Indicated for moderate low back pain, lumbar muscle weakness, lumbar sprains/strains
- > Manufactured from premium quality elastic with contact closure; includes both plastic and aluminum stays

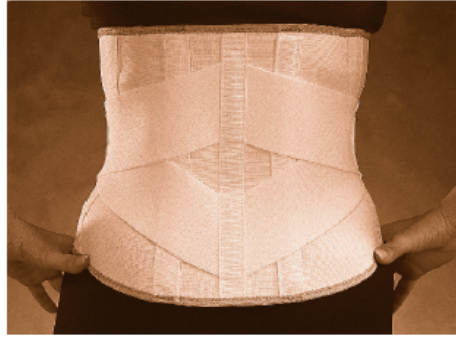

PRODUCT NUMBER

XS	S	M	L	XL
35-1760	35-1761	35-1762	35-1763	35-1764

2XL	3XL	4XL
35-1765	35-1766	35-1767

SIZING GUIDE

XS	S	M	L	XL
18"-24"	24"-30"	30"-36"	36"-42"	42"-48"

2XL	3XL	4XL
48"-52"	52"-56"	56"-60"

Measurement taken from hip circumference

APPLICATION INSTRUCTIONS

1. Open back support and lay flat.
2. Note: optional metal stays may be removed at this time from back panel if needed or to launder product.
3. Grasp side of support, making sure stay openings are facing upward and center on small of back.
4. Pull left side firmly across abdominal region, repeat for right side and secure loop portion onto hook portion.
5. Thread strap through buckle and secure..
6. Adjust to desired tension level, making sure not to over tighten.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.