



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



BACK SUPPORTS

BACK-2-FORM SUPPORT

- > Features tapered elastic side panel with elastic back panel and pocket that holds either a thermoplastic insert, gel pack, or pressure pad (sold separately); double-pull tension straps for added compression or web strap for static treatment
- > Indicated for moderate to severe low back pain, lumbar muscle weakness, lumbar strain or sprain, and lumbar instability
- > Manufactured from premium quality elastic with contact closure



Insert options:
37-6031 Thermoplastic Insert
77-9999 Pressure Pad
72-7006 Gel Pack

PRODUCT NUMBER

	XS	S	M	L	XL
	35-1840	35-1841	35-1842	35-1843	35-1844
W/Strap	35-1850	35-1851	35-1852	35-1853	35-1854

	2XL	3XL	4XL
	35-1845	35-1846	35-1847
W/Strap	35-1855	35-1856	35-1857

SIZING GUIDE

	XS	S	M	L	XL
	18"-24"	24"-30"	30"-36"	36"-42"	42"-48"

	2XL	3XL	4XL
	48"-52"	52"-56"	56"-60"

Measurement taken from hip circumference

APPLICATION INSTRUCTIONS

1. Lay back support flat, locate top of support by opening pocket (top).
2. If applicable, place gel pack, thermoplastic insert, or pressure pad into pocket if desired.
3. Grasp back support and center on small of back.
4. Pull left side firmly across middle section, repeat for right side. If applicable, attach double pull onto Velcro closure.
5. If product comes with suspenders, raise suspenders to shoulders and adjust to your height by sliding the buckle (suspenders are removable).

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.