



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



SPINAL ORTHOSIS

DISC UNLOADER ORTHOSIS

- > Unique modular design reduces intradiscal pressure through circumferential compression, standard 6" anterior and 12" posterior panels accept anatomically designed inserts along with gel pack that provides the benefits of cold/hot therapy
- > Indicated for chronic or acute low back pain, discogenic disc disease, post-surgical application, strains/sprains to lumbar muscles/ligaments and postural support
- > Constructed of soft breathable fabric with contoured rigid A.B.S. inserts, elastic double-pulls and contact closure



PRODUCT NUMBER

XS	S	M	L	XL
35-4000	35-4001	35-4002	35-4003	35-4004

2XL	GEL PACK
35-4005	35-4009

SIZING GUIDE

XS	S	M	L	XL
25"-30"	30"-35"	35"-40"	40"-45"	45"-50"

2XL	GEL PACK
50"-55"	10" x 10"

Measurement taken from waist circumference

APPLICATION INSTRUCTIONS

1. Release Velcro closures, open orthosis and lay flat.
2. Note: rigid inserts may be removed from pockets, heat molded to specifications, and reinserted or it may be used without inserts.
3. Place anterior (low profile) panel on center of abdomen, making sure loop portion is facing outward and curve is facing down.
4. Place posterior panel on center of back, making sure Corflex logo and elastic double-pulls are face outward.
5. Grasp both ends of posterior panel, bring forward, and attach to loop on anterior panel.
6. Grasp both ends of elastic double-pull, pull one side across and attach onto loop area on opposite side of posterior panel, pull other side across and attach to loop area on opposite double-pull.
7. Adjust to desired tension, making sure not to over tighten.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any plastic inserts if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:

TELEPHONE 1-800-426-7353, Ext1	FAX 1-603-623-4111	E-MAIL Service@corflex.com	MAIL CORFLEX, Inc. Attn: Customer Service 669 East Industrial Park Dr Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.