



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



WRIST SPLINTS

WRIST / HAND ORTHOSIS

- > Anatomically designed to conform to the natural shape of the wrist providing comfortable and secure immobilization
- > Indicated for immobilization of the wrist following post-op rehabilitation, stabilization of minor forearm and metacarpal fractures, post-cast removal or cast substitute for long-term use
- > Constructed of rigid polyethylene laminated to soft closed cell foam lining, includes two stockinette undersleeves



PRODUCT NUMBER

	XS	S	M	L	XL
Black Right	37-0500	37-0501	37-0502	37-0503	37-0504
Black Left	37-0510	37-0511	37-0512	37-0513	37-0514

	XS	S	M	L	XL
White Right	37-0600	37-0601	37-0602	37-0603	37-0604
White Left	37-0610	37-0611	37-0612	37-0613	37-0614

SIZING GUIDE

XS	S	M	L	XL
5"-6"	6"-7"	7"-8"	8"-9"	9"-10"

Measurement taken from wrist circumference

APPLICATION INSTRUCTIONS

1. Apply a stockinette undersleeve.
2. Open straps and slide onto thumb and/or hand.
3. Secure by closing straps to desired comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.