



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



THUMB SPLINTS

WRIST / HAND / THUMB ORTHOSIS

- > Anatomically designed to conform to the natural shape of the thumb and wrist providing comfortable and secure immobilization
- > Indicated for immobilization of the thumb for post-op rehabilitation, stabilization of thumb fractures, post cast removal, or cast substitute for long-term use
- > Constructed of rigid polyethylene laminated to soft closed cell foam lining, includes two stockinette undersleeves



PRODUCT NUMBER

	XS	S	M	L	XL
Black Right	37-0520	37-0521	37-0522	37-0523	37-0524
Black Left	37-0530	37-0531	37-0532	37-0533	37-0534

	XS	S	M	L	XL
White Right	37-0620	37-0621	37-0622	37-0623	37-0624
White Left	37-0630	37-0631	37-0632	37-0633	37-0634

SIZING GUIDE

XS	S	M	L	XL
5"-6"	6"-7"	7"-8"	8"-9"	9"-10"

Measurement taken from wrist circumference

APPLICATION INSTRUCTIONS

1. Apply a stockinette undersleeve.
2. Open straps and slide onto thumb and/or hand.
3. Secure by closing straps to desired comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE
1-800-426-7353, Ext1



FAX
1-603-623-4111



E-MAIL
Service@corflex.com



MAIL
CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.