



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## FINGER SPLINTS

### EXTENDED LENGTH BOXER SPLINT

- > Anatomically designed to provide comfortable support, protection and immobilization of the 4th and 5th metacarpals; extended length provides added wrist support
- > Indicated for treatment of acute or sub-acute fractures of the 4th and 5th metacarpals commonly referred to as "boxer fractures"
- > Constructed of lightweight low-density polyethylene shell with soft foam liner and palmer pad and contact closure straps



#### PRODUCT NUMBER

	S	M	L	XL
Right	37-1221	37-1222	37-1223	37-1224
Left	37-1231	37-1232	37-1233	37-1234

#### SIZING GUIDE

	S	M	L	XL
Wrist	4"-5"	5"-6"	6"-7"	7"-8"
4th & 5th MCP	2 3/4"-3 1/4"	3 1/4"-3 3/4"	3 3/4"-4 1/4"	4 1/4"-4 3/4"

Measurement taken from wrist circumference and 4th & 5th MCP joint

#### APPLICATION INSTRUCTIONS

1. Place splint on flat surface with opening facing up.
2. Open all straps and rest hand inside splint.
3. Make sure 4th and 5th metacarpals rest comfortably in angled portion of splint.
4. Secure distal strap around 4th and 5th metacarpals then secure proximal strap around wrist.
5. Slide middle strap through buckle, center pad across web space and secure strap snugly.
6. Retighten distal and proximal straps to desired comfort.

#### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

#### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.