



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HUMERAL SPLINTS

HUMERAL SPLINT

- > Provides circumferential soft tissue compression of the humerus while allowing full range of motion at shoulder and elbow
- > Indicated for treatment of distal or mid shaft humeral fractures
- > Constructed of rigid polyethylene laminated with closed cell foam lining and contact closure



PRODUCT NUMBER

S	M	L	XL
37-2061	37-2062	37-2063	37-2064

SIZING GUIDE

	S	M	L	XL
Proximal	9"-11"	11"-13"	13"-15"	15"-17"
Distal	7"-9"	9"-11"	11"-13"	13"-15"

Measurement taken at widest part of bicep and above elbow



APPLICATION INSTRUCTIONS

1. Slide arm into product and position.
2. Tighten straps and secure to desired level of comfort.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.