



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HUMERAL SPLINTS

EXTENDED LENGTH HUMERAL SPLINT

- > Controls humeral segments through circumferential soft tissue compression and features deltoid cap extension to prevent migration
- > Indicated for treatment of humeral diaphyseal fractures or deep humeral bone bruises
- > Constructed of rigid polyethylene laminated with closed cell foam lining and contact closure, includes two stockinette undersleeves and one arm sling



PRODUCT NUMBER

XS	S	M	L	XL	2XL
37-2160	37-2161	37-2162	37-2163	37-2164	37-2165

SIZING GUIDE

	XS	S	M	L	XL	2XL
Proximal	7"-9"	9"-11"	11"-13"	13"-15"	15"-17"	17"-19"
Distal	6"-7"	7"-9"	9"-11"	11"-13"	13"-15"	15"-17"

Measurement taken at widest part of bicep and above elbow

APPLICATION INSTRUCTIONS

1. Place one layer of stockinette on upper portion of arm.
2. Loosen all straps and slide over affected arm with deltoid cap resting comfortable on shoulder joint.
3. Gently tighten straps to achieve comfort, making sure not to over tightening straps.
4. If applicable, farthest end of splint may be trimmed and bottom strap moved up to allow 90 degrees of elbow flexion.
5. Criss cross long strap, attached to the closest end, over shoulder on injured side and pass through opposite underarm and back to buckle.

NOTE: Consult physician if swelling of lower arm and/or hand occurs.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.