



CORFLEX.
.....


**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



ELBOW SPLINTS

ELBOW ORTHOSIS W/R.O.M. HINGE

- > Anatomically designed humeral and ulnar shells with low-profile R.O.M. hinge provides comfortable, secure and effective range of motion control for the elbow
- > Indicated for use following elbow arthroplasty or other post-surgical or post-injury applications where limited range of motion is required
- > Constructed of rigid polyethylene laminated shells with closed cell foam lining and easy-to-set R.O.M. hinge; includes stockinette undersleeves

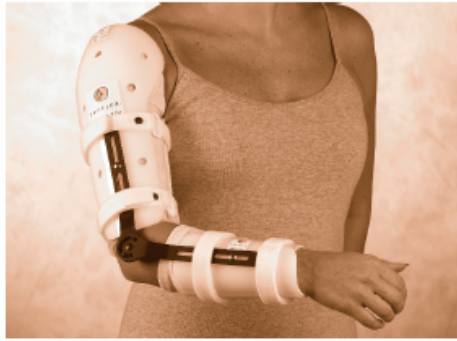

PRODUCT NUMBER

	S	M	L
Right	37-2201	37-2202	37-2203
Left	37-2211	37-2212	37-2213

SIZING GUIDE

HUMERAL	S	M	L
Proximal	9"-11"	11"-13"	13"-15"

ULNAR	S	M	L
Proximal	7"-9"	9"-11"	11"-13"

Measurements taken from widest part of bicep and forearm

APPLICATION INSTRUCTIONS

1. Place stockinette on upper (humeral) and lower (ulnar) portion of arm.
2. Loosen all straps and slide over affected arm with range of motion hinge aligned on outer (lateral) side of elbow joint.
3. Position humeral cuff on upper arm with deltoid cap resting comfortably on shoulder joint, making sure it allows for comfortable flexion of elbow joint (Note: farthest end of humeral cuff may be trimmed and bottom strap moved up if desired). Position ulnar cuff on lower arm, making sure not to limit wrist movement.
4. Hinge axis with condyle pad should be aligned with center of elbow joint and may be adjusted to appropriate arm length by moving hinge arms up or down on Humeral or Ulnar cuff.
5. Gently tighten straps, making sure not to over tighten.
6. Range of motion may be adjusted on hinge by setting pins to desired degrees of flexion/extension with wrench.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.