



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



ELBOW SPLINTS

VENTRAL CUBITAL TUNNEL SPLINT

- > Unique anterior design immobilizes and positions elbow at 122° while allowing easy access to dressing
- > Indicated for Cubital Tunnel Syndrome, post-op application
- > Constructed of high density polyethylene shell with plush cloth liner



PRODUCT NUMBER

S/M	M/L	L/XL
37-2291	37-2292	37-2293

SIZING GUIDE

S/M	Fits most females & smaller framed males
M/L	Fits most males & larger framed females
L/XL	Fits large framed males & XL framed females

APPLICATION INSTRUCTIONS

1. Locate top of splint as marked.
2. Apply device with arm flexed following the contour of the splint.
3. Attach distal strap first, followed by middle strap and then the proximal strap.
4. Excess hook may be trimmed once splint is applied satisfactorily.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.