



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## PLANTAR FASCIITIS SPLINTS

### Night Stretch Plantar Fasciitis Splint

- > Features polyethylene shell with soft liner and bi-lateral assist straps that allow patient to adjust splint to desired degree of dorsiflexion flexion
- > Indicated for treatment of plantar fasciitis, achilles tendonitis, plantar flexion contractures, heel spurs and other lower leg over- use injuries
- > Constructed of high density rigid shell with replaceable liner



#### PRODUCT NUMBER

	S	M	L	XL
Adjustable	37-4191	37-4192	37-4193	37-4194

#### SIZING GUIDE

	MENS SIZE	WOMENS SIZE
Small	5 1/2 - 6 1/2	5 - 6
Medium	7 - 8	6 1/2 - 7 1/2
Large	8 1/2 - 9 1/2	8 - 9
X-Large	10 - 11	9 1/2 - 10 1/2

Measurement by shoe size

#### APPLICATION INSTRUCTIONS

1. Apply device with the foot flexed to maximize the angle of dorsiflexion.
2. Attach instep strap first to hold the heel in the splint. Adjust length of strap and close the buckle.
3. Extend ankle to check the tightness of the instep strap.
4. After checking the fit of the instep strap, the other straps may then be adjusted and the buckles secured.
5. Bilateral dorsiflexion assist straps may then be adjusted to your tolerance level, if applicable. **DEVICE IS NON-WEIGHT BEARING.**

#### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

#### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.