



CORFLEX
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



PLANTAR FASCIITIS SPLINTS

PADDED PLANTAR FASCIITIS NIGHT SPLINT

- > Features polyethylene shell with soft padded cover and straps to provide maximum patient comfort; available in 90° fixed, 5° dorsiflexion and adjustable models
- > Indicated for treatment of plantar fasciitis, achilles tendonitis, plantar flexion contractures, heel spurs and other lower leg over-use injuries
- > Constructed of high density rigid shell, replaceable soft padded cover & contact closure



PRODUCT NUMBER

	S	M	L	XL	2XL
90° Fixed	37-4161	37-4162	37-4163	37-4164	37-4165
5° DF	37-4171	37-4172	37-4173	37-4174	37-4175
Adjustable	37-4181	37-4182	37-4183	37-4184	37-4185

SIZING GUIDE

	MENS SIZE	WOMENS SIZE
Small	5 ½ - 6 ½	5 - 6
Medium	7 - 8	6 ½ - 7 ½
Large	8 ½ - 9 ½	8 - 9
X-Large	10 - 11	9 ½ - 10 ½
2X-Large	11 ½ +	11 +

Measured by shoe size

APPLICATION INSTRUCTIONS

1. Apply device with the foot flexed to maximize the angle of dorsiflexion.
2. Attach instep strap first to hold the heel in the splint. Adjust length of strap and close the buckle.
3. Extend ankle to check the tightness of the instep strap.
4. After checking the fit of the instep strap, the other straps may then be adjusted and the buckles secured.
5. Bilateral dorsiflexion assist straps may then be adjusted to your tolerance level, if applicable. **DEVICE IS NON-WEIGHT BEARING.**

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.