



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



THUMB SPLINTS

LOW PROFILE THUMBSTER

- > Anatomically designed to provide comfortable and secure immobilization of the thumb while preserving mobility of the other fingers and wrist
- > Indicated for deQuervain's Syndrome, gamekeeper's thumb, tendonitis, thumb strains or sprains and post-cast removal
- > Manufactured from ultra-thin, rigid polyethylene with contact closure, also includes stockinette undersleeve



PRODUCT NUMBER

	S	M	L	XL
Right	37-5001	37-5002	37-5003	37-5004
Left	37-5011	37-5012	37-5013	37-5014

SIZING GUIDE

S	M	L	XL
6"-7"	7"-8"	8"-9"	9"-10"

Measurement taken from wrist circumference

APPLICATION INSTRUCTIONS

1. Place stockinette over affected thumb/hand.
2. Place thumb into splint and wrap strap around wrist and secure.
3. Fold stockinette down over splint.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.