



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



WRIST SPLINTS

ULTRA VINYL WRIST SPLINT

- > Features v-style loop locks with ultra thin web strap to provide secure immobilization of the wrist while preserving finger mobility
- > Indicated for Carpal Tunnel Syndrome, tendonitis, wrist strains or sprains, and post-cast removal
- > Constructed of premium grade perforated vinyl laminated to soft flannel lining with malleable aluminum palmer and dorsal stays and contact closures



PRODUCT NUMBER

	S	M	L	XL
8" Right	41-1521	41-1522	41-1523	41-1524
8" Left	41-1531	41-1532	41-1533	41-1534
10" Right	41-1541	41-1542	41-1543	41-1544
10" Left	41-1551	41-1552	41-1553	41-1554

	2XS	XS	S	M	L	XL
6" Right	41-1510	41-1513				
6" Left	41-1511	41-1514				

SIZING GUIDE

	2XS	XS	S	M	L	XL
	3"-4"	4"-5"	5"-6"	6"-7"	7"-8"	8"-9"

Measurement taken from wrist circumference



APPLICATION INSTRUCTIONS

1. Open wrist splint and lay flat (inside up) on any clean surface.
2. Place hand on splint so that the strap with the loop portion extends through the index finger and the thumb and attaches to hook on top of your hand.
3. Attach remaining straps by threading them through the buckles.
4. Pull tightly to slightly stretch the elastic holding the plastic buckles.
5. Attach all remaining straps starting at the wrist and working up the forearm.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.