



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



WRIST SPLINTS

AMBO ELASTIC WRIST SPLINT

- > Features reversible palmer stay that allows it to fit either left or right to help reduce inventory
- > Indicated for Carpal Tunnel Syndrome, tendonitis, and wrist strains or sprains
- > Manufactured from premium grade elastic with contact closure



PRODUCT NUMBER

	S	M	L	XL
6"	41-1751	41-1752	41-1753	41-1754
8"	41-1771	41-1772	41-1773	41-1774
10"	41-1791	41-1792	41-1793	41-1794

SIZING GUIDE

S	M	L	XL
5"-6"	6"-7"	7"-8"	8"-9"

Measurement taken from wrist circumference

APPLICATION INSTRUCTIONS

1. Detach straps from product.
2. Slide hand into product with top strap wrapping between thumb and index finger and secure closure.
3. Wrap the middle and bottom strap around and secure.
4. Depending upon which hand splint is being used on, make sure the curvature portion of palmer stay, is resting comfortably against palm of hand (below thumb).

Note: To make product fit right or left hand, remove palmer stay from opening of product located adjacent to widest strap, turn palmer stay over, and insert back into the product in the same opening as it was located before you removed.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.