



**CORFLEX.**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## ELBOW SPLINTS

### UNIVERSAL ELBOW IMMOBILIZER

- > Provides comfortable and secure immobilization of the elbow fixed at 90°
- > Indicated for trauma stabilization; fractures of the elbow, distal humerus or proximal ulna; and chronic olecranon bursitis
- > Constructed of foam laminated to pressure sensitive fabric with three rigid stays and bursa pad



#### PRODUCT NUMBER

S/M	L/XL
43-5071	43-5072

#### SIZING GUIDE

S/M	Fits most females & smaller framed males
L/XL	Fits most males & larger framed females

#### APPLICATION INSTRUCTIONS

1. Remove medial/lateral stays and open all straps.
2. Identify longer portion of brace and place forearm into unit, making sure elbow is seated firmly onto elbow pad. NOTE: elbow pad is removable if desired.
3. Grasp sides of shorter portion of brace and wrap around bicep area. Use small hook tab to assist in closure. NOTE: excess material may be trimmed at this time.
4. Grasp side of longer portion of brace and wrap around forearm area. Use small hook tab to assist in closures. NOTE: excess material may be trimmed at this time.
5. Place lateral 90 DEG stay assembly on brace, making sure stay falls at midline of arm.
6. Place medial 90 DEG stay assembly on brace, making sure stay falls at midline of arm.
7. Beginning with most distal strap, wrap contact closure around arm and secure. Then continue distal to proximal to secure all four straps.
8. Adjust straps to desired tension level, using caution not to over tighten.

#### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

#### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - [www.corflex.com](http://www.corflex.com)

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.