



CORFLEX


**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HAND / FOREARM SPLINTS

FOREARM SPLINT

- > Pre-molded design allows for optimal anatomic fit
- > Indicated for fractures of the wrist or distal forearm, severe sprains or strains of the wrist and post-operative rehabilitation
- > Constructed of malleable aluminum with open cell foam padding

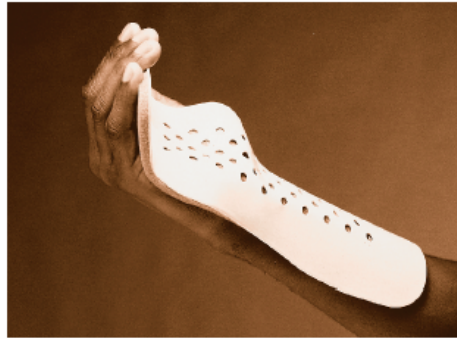

PRODUCT NUMBER

	XS	S	M	L
Right	49-2470	49-2471	49-2472	49-2473
Left	49-2570	49-2571	49-2572	49-2573

SIZING GUIDE

	XS	S	M	L
Length	6"	8"	10"	12"

APPLICATION INSTRUCTIONS

1. Select correct size.
2. Contour aluminum to individual patient based on condition.
3. Secure splint to hand with elastic bandage.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe exterior surface with damp cloth. You may use disinfectant or mild soap. Never use cleaning fluid, let pillow air dry, no not use any appliance for drying.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.