



CORFLEX.


**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



FINGER SPLINTS

CURVED FINGER SPLINTS

- > Lightweight design provides comfortable finger immobilization
- > Indicated for treatment of phalangeal and metacarpal fractures or dislocations
- > Constructed of malleable aluminum with foam padding

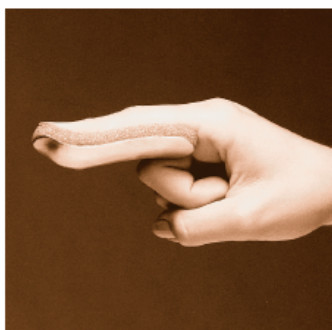

PRODUCT NUMBER

	1 1/2"	3"	4 1/2"	6"	9"
Pkg/12	49-3201	49-3202	49-3205	49-3203	49-3204

SIZING GUIDE

Length	1 1/2"	3"	4 1/2"	6"	9"

APPLICATION INSTRUCTIONS

1. Select correct size.
2. Contour aluminum to individual patient based on condition.
3. Secure splint to hand.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe exterior surface with damp cloth. You may use disinfectant or mild soap. Never use cleaning fluid, let pillow air dry, no not use any appliance for drying.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.