



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## WRIST SPLINTS

### CANVAS WRIST LACER SPLINT

- > Features cinch laces with contact closure for ease of application
- > Indicated for Carpal Tunnel Syndrome, tendonitis, and wrist strains or sprains
- > Constructed of durable canvas with soft flannel inner lining and malleable palmer stay



#### PRODUCT NUMBER

	XS	S	M	L	XL
8" Right	49-5040	49-5041	49-5042	49-5043	49-5044
8" Left	49-5140	49-5141	49-5142	49-5143	49-5144

	XS	S	M	L	XL
6" Right	49-5030	49-5031	49-5032	49-5033	49-5034
6" Left	49-5130	49-5131	49-5132	49-5133	49-5134

#### SIZING GUIDE

XS	S	M	L	XL
4"-5"	5"-6"	6"-7"	7"-8"	8"-9"

Measurement taken from wrist circumference

#### APPLICATION INSTRUCTIONS

1. Open straps.
2. Slide affected hand into wrist splint.
3. Attach strap through thumb and index finger.
4. Grasp hook attached to laces, wrap around and wrist and attach to loop portion.
5. Trim excess portion of strap if desired.

#### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

#### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.