



CORFLEX.


**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



KNEE MISC

ULTRA TRICOT KNEE IMMOBILIZER

- > Features sewn-in contoured posterior stays, medial/lateral stays for optimal immobilization
- > Indicated for post-op immobilization and mild to severe knee injuries where full extension of the leg is desired
- > Constructed of breathable foam with canvas outer, hook/loop closure straps, aluminum stays

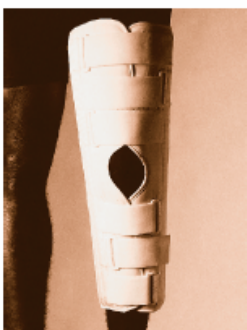

PRODUCT NUMBER

LENGTH	2XS	XS	S	M	L	XL
13"	51-1529	51-1530	51-1531	51-1532	51-1533	51-1534
17"		51-1540	51-1541	51-1542	51-1543	51-1544
19"			51-1551	51-1552	51-1553	51-1554
23"			51-1561	51-1562	51-1563	51-1564

SIZING GUIDE

	2XS	XS	S	M	L	XL
Width	7"-10"	10"-13"	13"-16"	16"-18"	18"-20"	20"-22"

Measurement taken from leg circumference 4" above mid-patella

APPLICATION INSTRUCTIONS

1. Open knee immobilizer and remove stay assemblies.
2. Position knee immobilizer so patella strap will go over the midline of the patella with wide end toward thigh and back stays along midline of leg.
3. Use small hook tab to hold body around knee.
4. Attach stay assemblies, inside first, to accommodate calf and thigh circumference.
5. Starting at the bottom and moving up, thread straps through buckles and pull back, attaching straps where comfortable.
6. Pull patella strap across knee and attach.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.