



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



KNEE IMMOBILIZERS

TRI PANEL KNEE IMMOBILIZER

- > Features movable medial/lateral stay panels and sewn-in contoured posterior stays to provide secure immobilization; universal design allows for inventory reduction
- > Indicated for post-op immobilization and mild to severe knee injuries where full extension of the leg is desired
- > Constructed of soft foam material with hook and loop closure straps and aluminum stays



PRODUCT NUMBER

LENGTH	13"	17"	19"	23"	26"
	51-7113	51-7117	51-7119	51-7123	51-7126

SIZING GUIDE

Fits up to 26" circumference

Measurement taken from leg circumference 4" above mid-patella

APPLICATION INSTRUCTIONS

1. Position knee immobilizer at the back of the leg, centering the rear stays mid-line along leg.
2. Adjust the movable side panels of the lateral portions of the knee, accommodating both thigh and calf circumference, and secure in place.
3. Starting at the bottom, thread straps through corresponding plastic buckles, and continue upward.
4. Secure to desired comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.