



CORFLEX.


**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HIP TRACTION

UNIVERSAL BUCKS TRACTION SPLINT

- > Universal design features medial and lateral stays for added support and convoluted foam to help prevent migration
- > Ideal for post-op treatment of hip fractures, hip dislocation and femur fractures
- > Constructed of convoluted foam with polyester pile, heel rest and aluminum spreader bar

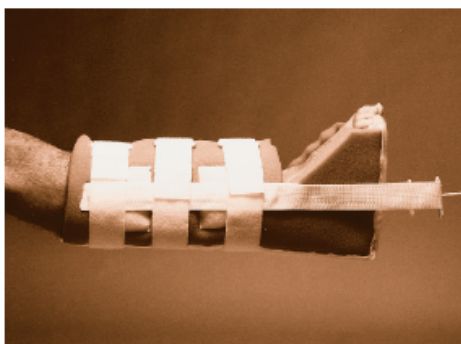

PRODUCT NUMBER

UNIV
54-2150

SIZING GUIDE

UNIV
12"-18"

Measurement taken at widest part of calf



APPLICATION INSTRUCTIONS

1. Release hook and loop closures.
2. Place leg in splint, making sure heel fits securely in distal end of splint and leg is properly aligned.
3. Adjust placement of stays as need. (Note: stays should be centered medially / laterally on leg).
4. Working from distal to proximal, secure hook and loop closures to desired tightness.
5. Affix to traction device (see manual specific to traction device).

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.