



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



SHOULDER MISC

HANGING CAST SLING

- > Features envelope style sling with contact closure
- > Indicated for control and reduction of edema in the hand and forearm, treatment of acromioclavicular sprains, anterior glenohumeral dislocation and elbow and wrist injuries
- > Manufactured from durable muslin material



PRODUCT NUMBER

UNIV
54-2170

SIZING GUIDE

UNIV
Width/Height 6" x 15"

Measurements are sling pouch width x height

APPLICATION INSTRUCTIONS

1. Release hook and loop closures.
2. Grasp sling by top section (longest end) and open.
3. Place arm in sling with forearm in top section, making sure elbow is secure in corner of the sling.
4. Secure hook and loop closures.
5. Affix to traction device.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.