



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



FOOT / ANKLE MISC

BUNION SPLINT

- > Designed to provide comfortable immobilization and alignment of the hallux
- > Indicated for post-operative alignment following hallux valgus surgery
- > Constructed of 1/8" soft foam laminated to flannel lining with movable stay and contact closure



PRODUCT NUMBER

	S	M	L
Right	61-5237	61-5238	61-5239
Left	61-5337	61-5338	61-5339

SIZING GUIDE

S	M	L
9"-10"	10"-11"	11"-12"

Measurement taken at widest part of foot

APPLICATION INSTRUCTIONS

1. Open splint and lay flat on clean surface with metal stay facing down.
2. Place big toe over stay area by small strap.
3. Grasp small strap, wrap around big toe, secure onto material and adjust to desired tightness.
4. Grasp large strap, wrap around foot, attach onto material and adjust to desired tightness.
5. Removable hook strip also included and may be used to further secure foot closure if desired.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.