



CORFLEX.

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



FOOT / ANKLE MISC

CLASSIC POST-OP SHOE

- > Features flat footbed with rocker-style bottom to promote a natural gait pattern with tongue closure for protection
- > Indicated for post-op foot protection and support
- > Constructed of corrugated rubber sole with water-resistant upper and contact closure



PRODUCT NUMBER

	S	M	L	XL	2XL
Mens	69-1022	69-1023	69-1024	69-1025	69-1029
Womens	69-1026	69-1027	69-1028		

SIZING GUIDE

LENGTH	S	M	L	XL	2XL
Mens	10"	11"	12"	13"	14"
Womens	9"	10"	11"		

APPLICATION INSTRUCTIONS

1. Place affected foot inside shoe.
2. Wrap the straps over the product onto opposite side, and attach straps onto hook portion of the product.
3. Straps should be pulled gently for comfortable fit.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.