



CORFLEX.
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



FOOT / ANKLE MISC

ENCLOSED TOE CAST BOOT

- > Features enclosed toe to provide complete protection and warmth for the foot
- > Indicated for lower extremity casted fractures
- > Constructed of ribbed rocker sole with water-resistant vinyl upper and contact closure



PRODUCT NUMBER

XS	S	M	L	XL
69-3100	69-3101	69-3102	69-3103	69-3104

SIZING GUIDE

	XS	S	M	L	XL
Length	7"	9"	11"	12"	14"



APPLICATION INSTRUCTIONS

1. Place affected foot inside shoe.
2. Wrap the straps over the product onto opposite side, and attach straps onto hook portion of the product.
3. Straps should be pulled gently for comfortable fit.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.