



**CORFLEX**  
.....

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



## SHOULDER MISC

### CRYOTHERM SHOULDER WRAP

- > Contours to shoulder, pockets hold gels packs to provide benefits of cold/hot therapy
- > Indicated for post-op use, treatment of arthritis or as part of a rehab program
- > Manufactured from soft breathable foam laminate with contact closure and pockets to hold reusable gel packs



#### PRODUCT NUMBER

0 GELS	2 GELS	4 GELS
72-7030	72-7032	72-7034

#### SIZING GUIDE

##### UNIV

Fits up to 48" chest circumference

Additional gels sold separately:  
 72-7001 - Single 5 1/2" x 11" Gel  
 72-7000 - Case/10 5 1/4" x 11" Gel

### APPLICATION INSTRUCTIONS

1. Observe warnings and instructions for use of gel packs.
2. Place one gel pack at a time into each pocket and secure in pocket.
3. Locate "neck" area of wrap and place on affected shoulder.
4. Place short strap under upper arm and attach to wrap in the front and back.
5. Locate longer strap and attach to front of wrap near neck area so strap comes across chest under the arm of the unaffected shoulder, across back and attached to the back of wrap.

### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

### HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**  
1-800-426-7353, Ext1



**FAX**  
1-603-623-4111



**E-MAIL**  
Service@corflex.com



**MAIL**  
CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.