



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**BACK MISC**

**CRYOTHERM BACK WRAP**

- > Anatomical design provides compression and concentrated cold/hot therapy for the back
- > Indicated for post-op use, arthritis, strains/sprains, and for integration into rehab program
- > Manufactured from a soft breathable foam laminate with contact closure and pockets to accommodate reusable gel packs and optional stays



**PRODUCT NUMBER**

	<b>0 GELS</b>	<b>2 GELS</b>	<b>4 GELS</b>
	72-7040	72-7042	72-7044
W/Stays	72-7040-300	72-7042-300	72-7044-300

**SIZING GUIDE**

**UNIV**

Fits up to 48" waist circumference

Additional gels sold separately:  
72-7001 - Single 5 1/2" x 11" Gel  
72-7000 - Case/10 5 1/2" x 11" Gel

**APPLICATION INSTRUCTIONS**

1. Observe warnings and instructions for use of gel packs.
2. Place one gel pack at a time into each pocket and secure in pocket.
3. Center product at small of back, pull left side firmly across middle section, repeat for the right side and secure in pocket.
4. Pull additional side pulls firmly and secure to desired comfort level.
5. If applicable, stay panels may be removed and/or adjusted for a comfortable fit.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.