



CORFLEX
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



TENNIS ELBOW SUPPORTS

CRYOTHERM ELBOW WRAP

- > Anatomical design provides compression and concentrated cold/hot therapy for the elbow
- > Indicated for post-op application, arthritis, strains or sprains of the elbow
- > Manufactured from a soft breathable foam laminate with contact closure and pocket to accommodate reusable gel pack



PRODUCT NUMBER

0 GELS	1 GEL	2 GELS
72-7050	72-7051	72-7052

SIZING GUIDE

UNIV

Fits up to 18" bicep circumference

Additional gels sold separately:
72-7003 - Single 3 1/2" x 11" Gel
72-7002 - Case/10 3 1/2" x 11" Gel

Measurement taken at widest part of bicep

APPLICATION INSTRUCTIONS

1. Observe warnings and instructions for use of gel packs.
2. Place one gel pack at a time into pocket and secure in pocket.
3. Locate top of wrap, slide arm through mesh supports, placing gel packs over affected area.
4. Working from bottom to top, wrap around and secure to desired comfort level.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.